

PARASITOLOGY CENTER, INC. (PCI)

11445 E. Via Linda, # 2-419

Scottsdale, AZ 85259

Ph. (480) 767-2522

Fax (480) 767-5855

[www.parasitetesting.com](http://www.parasitetesting.com)

---

## INSTRUCTIONS FOR PINWORM TEST

**Please read instructions completely before you begin.**

1. Holding the cap, remove paddle from the container. Do not touch the paddle.
2. Separate the buttocks and press the sticky surface of the paddle against the anal area. The specimen should be obtained early in the morning before arising or bowel movement.
3. Return the paddle to the container and secure cap tightly.
4. Wash hands thoroughly with soap after you have finished.
5. Print patient information on the label.
6. Store specimen in a cool place until mailing back to PCI for analysis.
7. Results will be made to your health care practitioner within 2-5 working days.

If you have any questions please use the “contact us” link on our website [www.parasitetesting.com](http://www.parasitetesting.com)

**You can receive your results directly from your patient profile; if you didn't register as a patient and only signed out as a guest, go to forget password, and then sign in, you will see your test under the tab reports.**